LipoAerosol®

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DE

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Directions for use

GB

LipoAerosol® Liposomal Suspension for application as an oral and throat spray or as an inhalation solution – preservative-free

Contains: Soy lecithin, sodium chloride, ethanol, water for injection

Areas of application:

LipoAerosol® is applied to humidify the upper and lower respiratory passages and to support the natural moisture film where the mucous membranes are irritated and for acute or chronic conditions of the respiratory passages.

As an oral and throat spray, LipoAerosol® can be used if your voice is hoarse and for vocal problems caused by infections of the upper respiratory passages, by straining your voice severely or by environmental irritations. In order to guarantee the best possible tolerance, LipoAerosol® contains no preservatives.

Please read the package leaflet carefully. This contains important information about the use of this medical device. If you have any questions, please see your doctor or pharmacist.

Dosage, method and duration of use:

Use as an inhalation solution


Use as required or according to your doctor's recommendation.

Generally, 2 to 4 times per day is recommended. Approximately 2 to 3 ml should be used for each inhalation.

LipoAerosol® is a ready-to-use inhalation solution. Do not combine LipoAerosol® with medications and/or other inhalation solutions.

Spray LipoAerosol® directly into the ready-to-use nebuliser:

- 16 sprays is equivalent to an amount of 2 ml
- 24 sprays is equivalent to an amount of 3 ml

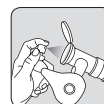LipoAerosol® should only be inhaled using a suitable nebuliser and atomiser / inhalation device. In principle, LipoAerosol® can be applied using a compression-driven jet nebuliser (e.g. PARI inhalation devices) or an ultrasonic nebuliser. The inhalation time may differ depending on the type of device used.

Please follow the device manufacturer's packaging information leaflet, in particular, the recommendations regarding cleaning, disinfection, hygiene, and care.

Use as a throat spray

1 to 3 sprays, several times per day as required, opening your mouth wide and spraying directly into your throat. A long, even breath should be taken while using the spray in order for the aerosol to be able to reach the larynx (e.g. for hoarseness). Hold the container as upright as possible and press the spray nozzle down fully. Avoid direct contact between the oral mucous membrane and the spray nozzle. The spray system creates an aerosol by atomising the solution which is suitable for use in the area of the upper respiratory passages.

Side effects:

In individual cases, the appearance of a temporary irritation (e.g. cough) or a reversible constriction of the bronchi with acute respiratory distress is conceivable, in particular, in patients with a hyper-responsive bronchial system (i.e. increased sensitivity and irritability of the respiratory passages, above all, following the inhalation of cold and/or moist air, cigarette smoke, perfume, etc.) e.g. in association with bronchial asthma. In this case, the inhalation should be ceased and medical attention should be sought if the discomfort persists. Do not use the spray if you have a tendency towards respiratory distress or hypersensitivity. A proprietary treatment with a medication which will dilate the bronchi may be recommended by your doctor.

Contraindications:

LipoAerosol® should only be used for patients with existing respiratory distress and/or narrowing of the bronchi due to moisture inhalation after consultation with the attending physician.

LipoAerosol® should not be used where there is hypersensitivity to one of its components.

The soya lecithin contained in LipoAerosol® consists of high-purity phospholipids, is protein-free and meets the quality standard for parenteral use.

Note:

In principle, when used as intended, there is no age restriction for the use of LipoAerosol® and no limitation regarding its use during pregnancy or while breast-feeding.

Seek medical aid if a severe sore throat with a high fever appears or where hoarseness persists for longer than three weeks.

The spray head can be exchanged for targeted application in the nebuliser or in the throat area. To do this, pull off the pre-mounted spray head and use light pressure to fit the supplied spray head with extended bar on the bar of the pump.

Packaging contents, appearance and fragrance:

Due to its liposomal formulation, LipoAerosol® is an opalescent liquid with a milky appearance and the characteristic natural fragrance of lecithin.

A brown-coloured opacity or residue may develop during storage, particularly in the area of the spring inside the pump unit. This is a characteristic of the vegetable component (soya lecithin), which has no effect on the effectiveness or tolerance, however.

Stability and storage:

Store in a cool, dry place away from light.

Keep out of reach of children.

Can be used for 6 months following the first use. Please note the date when first opened.

Do not use after the expiry date.

3 - 4 pumps of the spray may be required on first use in order to fill the vertical tube.

Information as at: 2021-09

Symbol/Symbol/Simbolo/Symbol:

	Hersteller	Fabriqué par	Prodotto da	Manufacturer
	Verfalldatum (JJJJ-MM: Jahr-Monat)	Date de péremption (AAAA-MM: Année-Mois)	Data di scadenza (AAAA-MM: Anno-Mese)	Expiry date (YYYY-MM: Year-Month)
	LOT-Nummer	Numéro de lot	Lotto numero	LOT-number
	Lagertemperatur 2°C - 25°C	Température de stockage 2°C - 25°C	Temperatura di conservazione 2°C - 25°C	Storage temperature 2°C-25°C
	Gebrauchsinformation beachten	Observer la notice explicative	Legerre attentamente il foglio illustrativo	Observe instructio leaflet
	Nicht verwenden wenn die Flasche beschädigt ist	Ne pas utiliser si le flacon est endommagé	Non usare se la bottiglia è danneggiata	Do not use if the bottle is damaged
	Nach Anbruch 6 Monate verwendbar.	A utiliser dans les 6 mois après ouverture.	Può essere usato fino a 6 mesi dopo il primo utilizzo.	Can be used for 6 month after first use.